FELIX^{RT}

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ENGLISH**INDEX**

1. Preface	46
2. Safety	46
3. Safety rules	48
3.1 General	48
3.2 Product safety symbols	49
3.3 Modification	52
3.4 Safety check	52
3.5 Weight capacity	53
3.6 Transporting your power chair	53
3.7 Incline guidelines	53
3.8 Outdoor operation & inclement weather precautions	54
3.9 Physical limitations	54
3.10 Emi information	55
3.11 Fireproof performance	55
4. Your product	56
4.1 Manual freewheel lever	57
4.2 Battery Gauge	58
4.3 Power button/status indicator	58
4.4 Drive function selector	58
4.5 Seat function selector	59
4.6 Hazard lights	59
4.7 Indicator lights	59
4.8 Position lights	60
4.9 Speed dial	60
4.10 Horn button	60
4.11 Joystick	60
4.12 Anti-tip wheels	61
5. Operation	62
5.1 Before getting onto your power chair	62
5.2 Getting onto your power chair	62
5.3 Pre-Ride Adjustment and check	62
5.4 Operating your power chair	62
5.5 Getting off of your power chair	63
6. Comfort adjustment	64
6.1 operation console adjustment	64
6.2 Armrests height and width adjustment	64
6.3 Footrests adjustment	65
6.4 Backrest adjustment	67
6.5 Seat tilting adjustment	67
6.6 Headrest adjustment	68

7. Basic troubleshooting	69
8. Batteries and charging	71
8.1 First charge	71
8.2 Charging your power chair	71
8.3 Operating temperatures	71
8.4 Battery operation and charging	72
8.5 Charging batteries	72
9. Care and maintenance	73
9.1 Tire	73
9.2 Wheel replacement	73
9.3 Exterior surfaces	73
9.4 Cleaning and disinfection	74
9.5 Battery terminal connections	74
9.6 Wiring harnesses	74
9.7 Daily checks	74
9.8 Weekly checks	74
9.9 Monthly checks	75
9.10 Yearly checks	75
9.11 ABS plastic covers	75
9.12 Axle bearings and motor/transaxle assembly	75
9.13 Console, charger, and rear electronics	75
9.14 Nylon lock nut replacement	75
9.15 Storing your power chair	75
9.16 Disposal of your power chair	76
10. Specifications	77
11. Warranty	79
Appendix I: EMC Compliance Information	80

1. PREFACE

Congratulations! You are now the owner of a new Power Chair from Loh Medical®! Thank you for choosing us to help improve your mobility and independence.

This Power Chair is made by qualified and committed personnel. It is designed and produced according to high-quality standards guarded. Please read this Owner's Manual before operating your Power Chair for the first time. Improper use of the Power Chair may result in harm, injury, or traffic accidents.

If you still have questions after reading this manual, do not hesitate to contact your specialist dealer. They will be glad to help you on this subject.

WARNING!

Improper use could lead to death, serious injury, hazardous conditions, or product malfunction.

ATTENTION!

Improper use could lead to injury and/or damage to your Power Chair.

SUGGESTION!

Follow these instructions for best practices, and to keep your Power Chair in prime operating order.

If someone else uses the Power Chair, please provide a copy of the Owner's Manual for their consideration.

This Owner's Manual is composed of the product design and specifications at the time of publication. As design changes, some illustrations and pictures in the manual may not correspond to the Power Chair you purchased. We reserve the right to make design modifications.

The manufacturer disclaims all responsibilities for any personal injury or property damage that may occur due to improper or unsafe use of this product.

2. SAFETY NOTICE

- Be aware the device is a small Power Chair and is designed mainly for indoor use. Extreme care should be taken when cornering and negotiating drops, obstacles, and gradients.

- Regarding to the tilting and reclining adjustment function of the seat, do not operate the adjustment mechanism while somebody is sitting on the seat.
- Read and follow the information in the owner's manual. Do not use this product or any available optional equipment without first thoroughly reading and understanding these instructions. If you cannot understand the warnings, cautions, or instructions, contact your dealer before attempting to use this equipment. Otherwise, injury or damage may occur. Never try to use your Power Chair beyond its limitations, as described in this manual.
- Keep your feet on the footboard at all times during operation. Do not stand on the footboard.
- Only drive on flat surfaces where all four wheels touch the ground and have sufficient contact to operate the Power Chair safely.
- If the battery gauge drops to red, charge the Power Chair as soon as possible.
- Completely discharging the batteries may damage the batteries or shorten the expected life of the batteries.
- Keep metal objects away from the battery terminals. Electric shock may occur.
- Always reduce your speed and maintain a stable center of gravity when cornering. Do not corner sharply when driving Power Chairs at higher speeds, and reduce speed before cornering.
- Make sure that there are no obstacles behind you while reserving your Power Chair.
- Do not allow children to play near or operate the Power Chair.
- Some of the parts of the Power Chair are susceptible to change in temperature. The controller can only operate in temperature that ranges between -10°C ~ 50°C (-14°F to 122°F).
- At extremely low temperatures, the batteries may freeze, and your Power Chair may not be able to operate. In extremely high temperatures, it may operate at slower speeds due to a safety feature of the controller that prevents damage to the motors and other electrical components.
- Store in a clean and dry condition.

- Do not drive your Power Chair at night without proper lighting.
- Do not remove the anti-tip wheels or transit wheels.
- Keep your hands and feet away from moving parts while driving. Be aware of loose-fitting clothes that can become caught in the drive wheels.
- Surface temperatures can increase when exposed to external sources of heat.
- Do not connect an extension cord to the battery charger.
- Disassembling the controller, motor, or charger by anyone other than an authorized service agent is prohibited and voids any applicable warranty.
- Do not take your Power Chair on roads or highways.
- Do not use your Power Chair if you are under the influence of alcohol, medicines, or other substances that may influence your driving abilities.
- Be cautious when driving your Power Chair in busy areas or shopping malls.
- Under no circumstances should the Power Chair be used as a seat in a motor vehicle.
- Do not attempt to lift your Power Chair by any parts other than the frame (for example, seat or body shrouds).

3. SAFETY GUIDELINES

3.1 General

Before operating your Power Chair, fully read and understand this Owner's Manual.

Users may encounter difficult maneuvering situations such as narrow doorways, traveling up and down ramps, cornering, and traveling on uneven terrain. Be sure to lower the speed, take your time, and carefully maneuver the Power Chair.

3.2 Product safety symbols

The symbols below are used on the Power Chair to identify warnings, mandatory actions, and prohibited actions. It is very important for you to read understand them completely.

 Power Chair information label.

DOES NOT meet ISO 7176-19 standards for occupied transport in a motor vehicle. When travelling in a motor vehicle, do not sit in your Power Chair.

Fully charge batteries before operating.
Remove key from an unattended Power Chair.

Powered wheelchair is not suitable for use as a seat for an occupant when in a motor vehicle.


Armrest width adjustment MAX POSITION When sliding the armrests inwards to adjust the width, it must not exceed this position to prevent the armrests from hurting the seat.

Manufactured.

Contact your local recycling center or authorized provider for information on proper recycling of product components.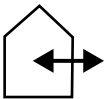

Indicates component is for indoor/outdoor use (battery charger excluded).

Contains Lead.

Your product must be disposed of according to applicable local and national statutory regulations. Contact your local recycling center or authorized provider for information on proper disposal of product components.

Indicates a potentially hazardous situation which, if not avoided, could result in death or severe injury. Property damage may also result.

Indicates electrical shock hazard.

EMI-RFI - This product has been tested and passed at an immunity level of 20 V/m.

Indicates that explosive conditions exist.

Corrosive chemicals contained in batteries.

Pinch/Crush points exist.

Indicates a mandatory action that should be performed as specified in order to avoid a hazardous condition/situation. Failure to perform mandatory actions can lead to personal injury and/or equipment damage.

Read and follow the information in the owner's manual.

Indicates maximum weight capacity. Stay within the specified weight capacity of your product.

Use only AGM or Gel-Cell batteries to reduce the risk of leakage or explosive conditions.

Wear safety goggles.

N = Neutral (Brakes disengaged; unit in Freewheel Mode)

D = Drive (Brakes engaged; unit in Drive Mode)

Indicates a prohibited action that should not be performed at any time or in any circumstances. Performing a prohibited action can cause personal injury and/or equipment damage.

Do not use batteries with different amp-hour (Ah) capacities. Do not mix old and new batteries. Always replace both batteries at the same time.

Do not mix AGM and Gel-Cell batteries. Always use batteries of the same type and chemistry.

Keep tools and other metal objects away from battery terminals. Contact with tools can cause electric shock.

Do not allow unsupervised children to play near the Power Chair while the batteries are charging.

Removal of grounding prong can create electrical hazard. If necessary, properly install an approved 3-pronged adapter to an electrical outlet having 2-pronged plug access.

Do not connect an extension cord to the AC/DC converter or the battery charger.

Indicates flammable material. Do not expose to heat sources such as open flame or sparks. Do not transport batteries with flammable or combustible items.

Avoid exposure to rain, snow, ice, salt, or standing water whenever possible. Maintain and store in a clean and dry condition.

Do not remove anti-tip wheels.

Keep your hands away from the tires when driving. Be aware that loose fitting clothing can become caught in the drive tires.

Do not engage freewheel mode on an incline.

Do not engage freewheel mode on a decline.

24V 38AH Battery capacity label

24V 52AH Battery capacity label

3.3 Modification

Do not modify, remove, disable, or add any parts, features, or functions on your Power Chair. We offers a wide variety of accessories for your Power Chair to help accommodate your needs.

WARNING – Do not modify your Power Chair in any way; doing so will void the warranty.

3.4 Safety check

ATTENTION – Inspect the following components before each use:

- Electrical connections and wire harness – secure and free from corrosion.
- Batteries – fully charged and free from corrosion.
- Brake – check the electromagnetic brake is functioning.

- Folding system – ensure the Power Chair is fully folded out.
- The condition of the tires – make sure they are not damaged or excessively worn. If you notice a problem, please contact your authorized Provider for assistance.

3.5 Weight capacity

This Power Chair is designed and produced solely to transport one person with a maximum weight of 136kg. It is not designed to transport goods or objects or use other than the transport previously described.


WARNING – Exceeding the maximum weight capacity will void your warranty. We will not be held responsible for injury or damages resulting from failure to observe weight capacities.

3.6 Transporting your Power Chair

When transporting your powered wheelchair by vehicle, it should be securely stowed in the back of a van or truck. Adjustable parts should be removed or properly secured during transport. Your power chair is suitable for land and maritime transport.

In order to ensure the safety of the user and to ensure that the powered wheelchair is not damaged during transport, securement-strap must be hooked to the securement points (A) and (B) on the wheelchair following the supplier instructions of the strap-system.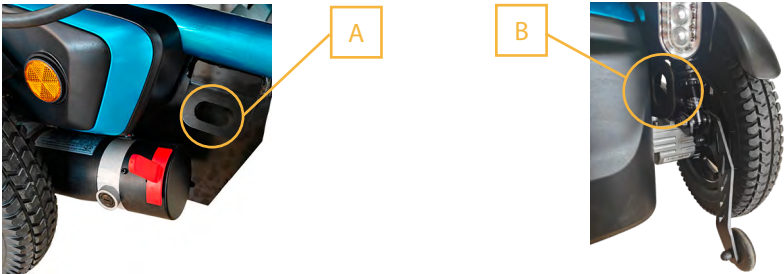

Do not sit in your Power Chair while in a moving vehicle.

3.7 Incline guidelines

Use caution when approaching inclines or declines; if necessary lower the speed before traveling on a slope. When traveling up an incline, you need to try to keep the Power Chair moving forward. If you must come to a stop, use caution and slowly accelerate the Power Chair forward.

When traveling down an incline, lower the speed on your Power Chair to the lowest setting, and proceed cautiously.

If the Power Chair is traveling down the incline faster than you expected, slowly release the throttle to come to a stop, then slightly push the throttle forward to continue safely down the incline.

WARNING – Drive your Power Chair straight up or down an incline, never backward. Erratic movements or back and forth patterns may increase the chance of tipping. Changes in inclines may decrease stability. Under any circumstances, do not travel on a slope greater than the maximum stability angle for this Power Chair.

3.8 Outdoor operation & inclement weather precautions

While your Power Chair is purposed for in-home use to assist you in daily living activities, we understands there are times you need to operate the Power Chair outdoors and has designed it to perform superbly on level outdoor surfaces.

ATTENTION – When operating outdoors, avoid uneven terrain, soft surfaces, tall grass, loose gravel, loose sand, freshwater, saltwater, edges of streams, lakes, or oceans. If you are unsure about a surface, avoid it.

Exposure to inclement weather should always be avoided. If you find yourself caught in inclement weather while operating your Power Chair, proceed to the closest shelter immediately. Completely dry your Power Chair before operating, charging, or storing.

3.9 Physical limitations

This Power Chair is designed and produced solely to transport one person with a maximum weight of 136 kg, and the user must have good eyesight, Lucid consciousness. You must exercise awareness, caution, care, and common sense when operating your Power Chair.

Always keep in mind your own limitations, and substance use when operating your Power Chair.

WARNING – Never use your Power Chair while under the influence of alcohol or mind-altering substances. Never smoke while seated in your Power Chair. Damaged or worn upholstery increases the risk of fire hazards and should be replaced immediately.

ATTENTION – Be aware of precautions, warnings, and safety issues when taking prescribed or over-the-counter drugs while operating your Power Chair.

3.10 EMI information

Electromagnetic interference (EMI) tests have been shown to produce adverse effects on the performance and control of electrically powered mobility devices.

EMI can be produced from different sources such as cellular phones, two-way radios, radio stations, television stations, amateur radio transmitters (HAM), wireless computer links, microwave signals, paging transmitters, and transceivers used by emergency vehicles.

The EMI waves can cause unintentional movement of the Power Chair or damage to the controller. Every electrically-powered mobility device has a resistance to EMI.

The higher the resistance level, the greater the protection.

The intensity of the interference can be measured in volts per meter, V/m. At this time, current technology is capable of achieving an immunity level of 20 V/m, which provides protection from more common sources of EMI.

Your Power Chair, with no modifications, has an immunity level of 20 V/m.

EMI becomes more intense as you move closer to the source. The EMI produced from hand-held radios is of special concern. It is possible to unintentionally bring high levels of EMI very close to the Power Chair's control system, affecting your movement and braking.

The warnings listed below are recommended to prevent possible interference with the control system of your Power Chair.

WARNING – Do not operate hand-held transceivers such as CB (citizens band) radios, or turn on personal communication devices, such as cellular phones, while the Power Chair is powered on.


WARNING – Be aware of nearby transmitters, such as radio or television stations, and avoid close.

3.11 Fireproof performance

Ignitability of upholstery according to EN1021-2.

4. YOUR PRODUCT

The Power Chair is easy to operate and is primarily intended for indoor use. The following picture shows all parts that are relevant for the user/driver. In the operation and maintenance section, these parts are described if applicable. The separable Power Chair has the brakes on top of the rear cover.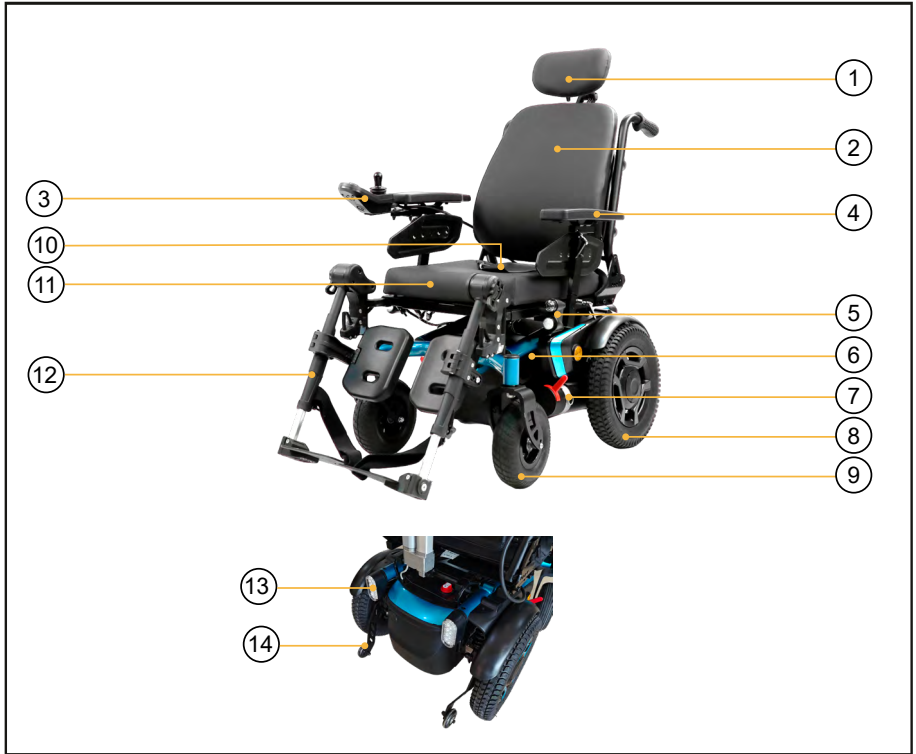


- | | |
|------------------------|-----------------------------|
| 1. Headrest | 8. Rear wheels/Drive wheels |
| 2. Backrest | 9. Front wheels |
| 3. Control console | 10. Seat belt |
| 4. Armrest | 11. Seat |
| 5. Headlights | 12. Footrest |
| 6. Battery compartment | 13. Rear lights |
| 7. Drive motors | 14. Anti-tip wheels |


4.1 Manual freewheel lever

Whenever you need or want to push your Power Chair for short distances, you can put it in freewheel mode.

1. Locate the manual freewheel lever at the top right of the rear frame.
2. Push forward on the manual freewheel lever to disable the drive system and the brake system. You may now push your Power Chair.
3. Push the manual freewheel lever rearward to reengage the drive and the brake systems; this takes your Power Chair out of freewheel mode.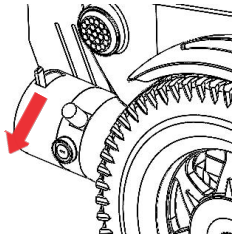

Drive Mode

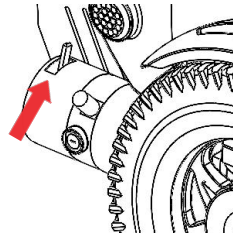Freewheel Mode

WARNING – Before placing your Power Chair into or taking it out of freewheel mode, remove the key from the key switch. Never sit on a Power Chair when it is in freewheel mode. Never put a Power Chair in freewheel mode on any incline. When your Power Chair is in freewheel mode, the braking system is disengaged.

- Disengage the drive motors only on a level surface.
- Ensure the key is removed from the key switch.
- Stand to the side of the Power Chair to engage or disengage freewheel mode.
- Never sit on a Power Chair to do this.
- After you have finished pushing your Power Chair, always return it to the drive mode to lock the brakes.

1. Battery gauge
2. Power button (Status indicator)
3. Drive function selector
4. Lights (hazard lights and left indicator)
5. Speed dial
6. Seat function selector
7. Horn button
8. Lights (position and right indicator)
9. Joystick

4.2 Battery Gauge

When the Power button is pressed down to power up your power chair, this gauge indicates approximate battery strength. For further information on the battery gauge, see 8. "Batteries and Charging."

4.3 Power button/Status indicator

Power On

Power Off

To switch ON the Power Chair, press the power button. The power button is the only user input that can activate the system of wheelchair.

- If there is no fault with the system, the status indicator (through the power button) will light up green, and the battery gauge will display the current battery status.
- If there is a fault with the system when powering up, the status indicator will indicate the fault with a series of red flashes, do not hesitate to contact your specialist dealer. They will be glad to help you on this subject.

To switch OFF the Power Chair, press the power button; the system will power down and the status indicator will switch off.

4.4 Drive function selector

Drive function select button and display

The wheelchair's drive function can be selected with the drive function select button.

- Press the top of the button to select the next drive function.
- Press the bottom of the button to select the previous drive function.

The drive function selected is indicated on the drive function indicator, which is located to the right-hand side of the drive select button. The indicator has three LEDs:

- when drive function 1 is selected, the bottom indicator will be lit, meaning the current maximum speed is on "High" level.
- when drive function 2 is selected, the bottom and middle indicators will be lit, meaning the current maximum speed is on "Medium" level.
- when drive function 3 is selected, all indicators will be lit, meaning the current maximum speed is on "Low" level.

4.5 Seat Function selector

Seat function select button

A seating function for activating motions can be selected with the seating function select button.

- Press the top of the button to select the next seating function.
- Press the bottom of the button to select the previous seating function.

WARNING – only seating functions that have been programmed will be available for selection.

The selected seating function is displayed on the drive/actuator status indicator as shown below.

Seating function	Display	Seating function	Display
Tilt 0° to 20°		Recline 90° to 140°	

4.6 Hazard lights

- To switch on the hazard lights, press and quickly release the left-hand lighting button once. LEDs behind the left-hand button and right-hand button will flash on and off for the duration the hazard lights are operating.
- To switch off the hazard lights, press the left-hand or right-hand lighting button.

4.7 Indicator lights

- To switch on the left indicator, press and hold the left-hand lighting button once. The LED behind the left-hand button will flash on and off for the duration the left indicator lights are operating.


- To switch on the right indicator, press and hold the right-hand lighting button once. The LED behind the right-hand button will flash on and off for the duration the right indicator lights are operating.
- To switch off either indicator, press the left-hand or right-hand lighting button.

4.8 Position lights

- To switch on the position lights, press the right-hand lighting button once.
- The LED behind this button will remain on for the duration the position lights are operating.
- To switch off the position lights, press the button again.


4.9 Speed dial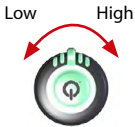

Speed dial

The speed dial allows the user to limit the maximum speed of the power chair (that is, the speed when the joystick is fully deflected) to suit their preference and environment.

The dial offers 10 discrete steps between the lowest speed (dial set to the left) and the highest speed (dial set to the right).

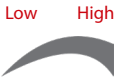As a visual reminder, a speed symbol (shown right) is positioned by the speed dial to indicate the low and high positions of the speed dial.

4.10 Horn Button

This button activates a warning horn. Your Power Chair must be turned on for the horn to be operational. Press the horn button to sound the horn.

The horn will sound for as long as the horn button is pressed. Do not hesitate to use the warning horn when doing so may prevent accident or injury.

4.11 Joystick

The joystick controls the direction and speed of drive and seating functions.

Direction control -driving

When the joystick is deflected from the neutral position, the wheelchair will move in the same direction as the joystick.

Direction control-seating

For seating functions, user has to select the actuator first by using the Seat function select button. When the joystick is deflected forward from the neutral position, the seating motion of selected actuator will extend. When the joystick is deflected backward from the neutral position, the seating motion of selected actuator will retract.


Stopping

In general, to stop driving or to stop a seating motion, either pull the joystick back to the neutral position, or release the joystick and it will automatically return to the neutral position.

4.12 Anti-tip wheels

The anti-tip wheels of this Power Chair has good performance and strong safety. The design of the anti-tip wheels enhances the stability of the wheelchair.

This reduces the likelihood of the wheelchair tipping over during the uphill process.

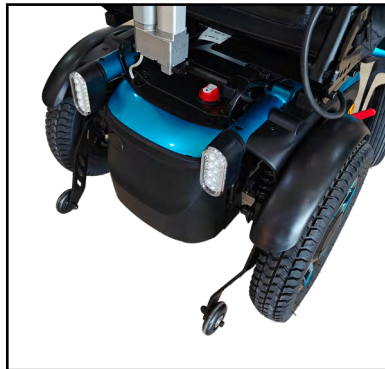5. OPERATION

5.1 Before getting onto your Power Chair

- Have you fully charged the batteries? See 9. "Batteries and Charging."
- Is the manual freewheel lever in the drive (outward) position? Never leave the manual freewheel lever in the inward position unless you are manually pushing your Power Chair.
- Ensure the air temperature is between -13°F/-25°C to 122°F/50°C.
- Ensure the air humidity is between 30% to 70%.
- Ensure the atmospheric pressure is between 86 kPa to 106kPa.

5.2 Getting onto your Power Chair

1. Make certain that the power button is in the status of power off .

WARNING – Never attempt to get onto or off of your Power Chair without first powering off your wheelchair. This will prevent the Power Chair from moving if accidental joystick contact is made.

2. Stand at the side of your Power Chair.
3. Disengage the footrest lever and rotate both footrests outwards until the seat is freely to walk in.
4. Position yourself comfortably and securely in the seat.
5. Rotate both footrests inwards until you can put your feet on them comfortably.
6. Make certain that your feet are safely on the footplates.
7. Make certain that your Safety belt is fastened.

5.3 Pre-Ride Adjustment and check

- Is the seat at the proper sitting position? See 6. "Comfort Adjustments."
- Is the seat secured into place?
- Is the power button pressed down and the status indicator (through the power button) light up green?
- Does the Power Chair's horn work properly?
- Is your proposed path clear of people, pets, and obstacles?
- Have you planned your route to avoid adverse terrain and as many inclines as possible?

5.4 Operating your Power Chair

WARNING – The following can adversely affect steering and stability while operating your Power Chair:

- Holding onto or attaching a leash to walk your pet.
- Carrying passengers (including pets).
- Hanging any article from the armrests or operating console.
- Towing or being pushed by another motorized vehicle.

WARNING – Keep both arms on the armrests and your feet on the footplates at all times while operating your Power Chair. This driving position gives you the most control over your vehicle.

- Set the speed adjustment dial to your desired speed.
- Press your fingers against the appropriate side of the joystick.
- The electromechanical disc park brake automatically disengages, and the Power Chair accelerates smoothly to the speed you preselected with the speed adjustment dial.
- Push the joystick in the direction of front left to steer your Power Chair to the left.
- Push the joystick in the direction of front right to steer your Power Chair to the right.
- Push the joystick forward to drive straight ahead.
- To stop, slowly release the joystick. The electronic brakes will automatically engage when your Power Chair comes to a stop.

NOTE: Your Power Chair's reverse speed is slower than that of the forward speed you preset with the speed adjustment dial.

5.5 Getting off of your Power Chair

1. Bring your Power Chair to a complete stop.

WARNING – Never attempt to get onto or off of your Power Chair without first powering off the wheelchair with the power button. This will prevent the Power Chair from moving if accidental joystick contact is made.


2. Press down the power button to power off.
3. Loosen the safety belt.
4. Disengage the footrest lever and rotate both footrests outwards until you are free to walk out.
5. Carefully and safely, get out of the seat and stand to the side of your Power Chair.
6. Rotate both footrests inwards until they restore to the original positions.

6. COMFORT ADJUSTMENT

6.1 Operation Console Adjustment

It is easy to adjust the operation console:


1. Rotate the screw counterclockwise.
2. Loosen the controller adjustment tube.
3. Pull the controller adjustment tube forward or backward to adjust to the desired position.
4. Rotate the screw clockwise to tighten the controller adjustment tube.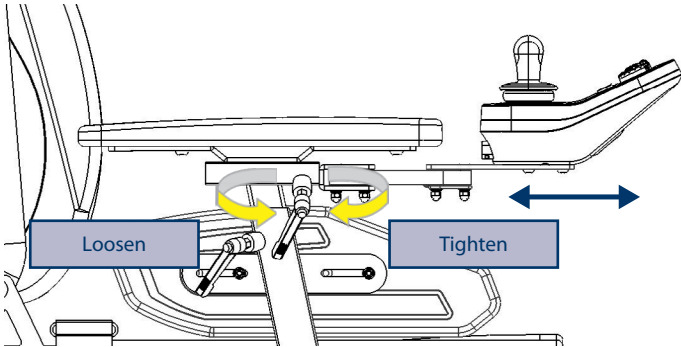

6.2 Armrests height and width Adjustment Width Adjustment

It is easy to adjust the armrests width:


1. Using the tools that come with the wheelchair to rotate the 2x screws counterclockwise.
2. Loosen the armrest width adjustment tube.
3. Pull the armrest outward or inward to adjust.
4. Repeat step 1-3 to the armrest of the other side.
5. Check both armrests if they are set to the desired positions.
6. Rotate the screws clockwise to tighten the armrest width adjustment tubes.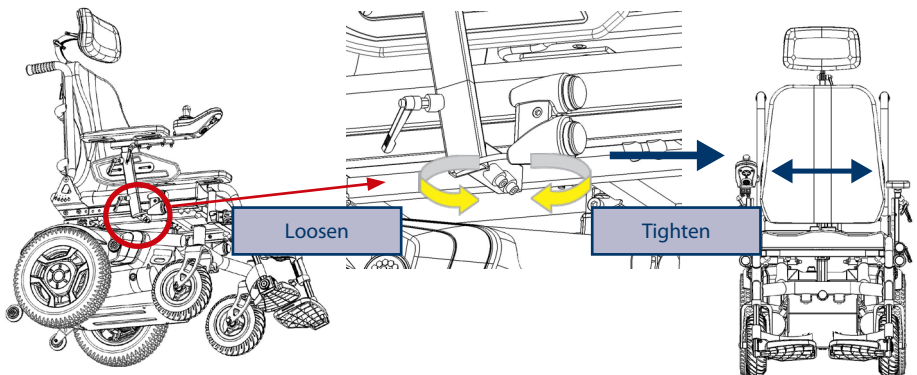

Height Adjustment

It is easy to adjust the armrests width:


1. Rotate the 2x screws counterclockwise.
2. Loosen the armrest height adjustment tubes.
3. Pull the armrest upward or downward to adjust to the desired position;
4. Rotate the screws clockwise to tighten the armrest height adjustment tube.
5. Repeat step 1-4 to the armrest of the other side.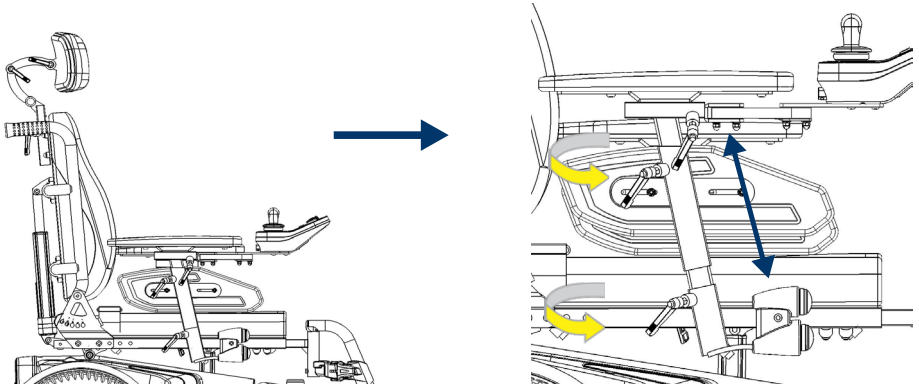

6.3 Footrests Adjustment Position Adjustment

It is easy to adjust the footrests position:


1. Using the tools that come with the wheelchair to rotate the 2x screws counterclockwise.
2. Loosen the footrest adjustment tube.
3. Pull the footrest outward or inward to adjust.
4. Repeat step 1-3 to the footrest of the other side.
5. Check both footrests if they are set to the desired positions.
6. Rotate the screws clockwise to tighten the footrests adjustment tube.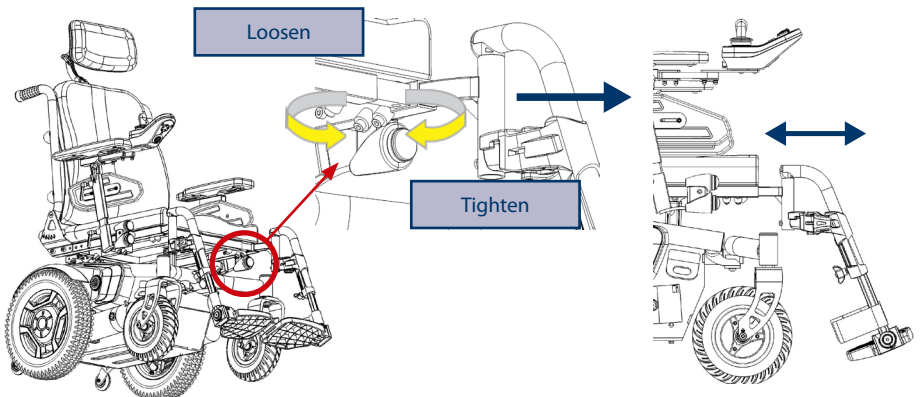

Remove footrests

It is easy to remove the footrests:


1. Flip up the footplate.
2. Pull the handle outward to release the locking mechanism.
3. Rotate the footrest outward.
4. Let go of the handle and pull up the footrest to remove.
5. Repeat step 1-4 to the footrest of the other side.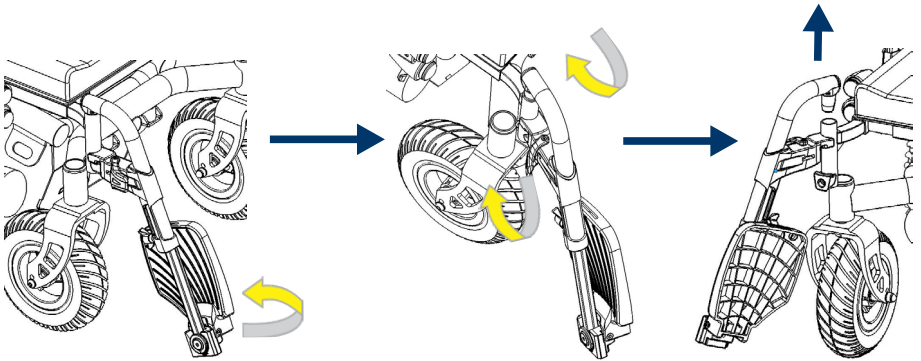

Footplates height Adjustment

It is easy to adjust the footplates height:

1. Rotate the adjustment knob counterclockwise.
2. Loosen the footplate height adjustment tubes.
3. Pull the footplate upward or downward to adjust to the desired position;
4. Rotate the adjustment knob clockwise to tighten the footplate height adjustment tube.
5. Repeat step 1-4 to the footplate of the other side.

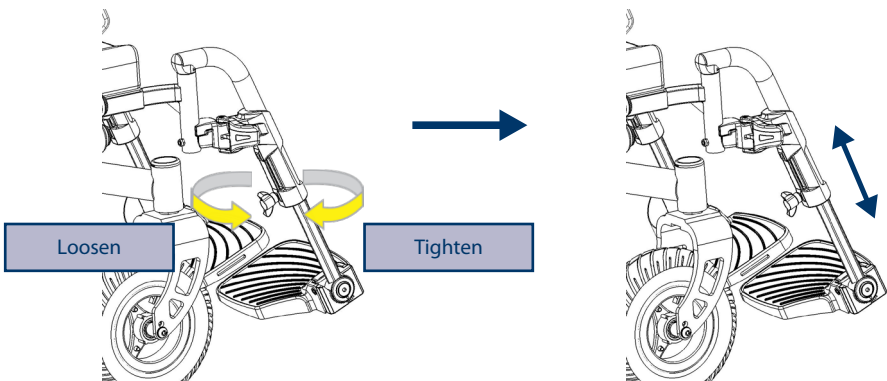6.4 Backrest Adjustment Reclining Actuator

If your wheelchair is equipped with reclining actuator. It is easy to adjust the backrest position 90° to 140°:

1. Press the seat function select button to choose the "Recline" seating function.
2. The selected seating function will be displayed on the drive/actuator status indicator.
3. Pull the joystick forward or backward from the neutral position.
4. The reclining angle of backrest will change accordingly.

6.5 Seat tilting Adjustment Tilt Actuator

If your wheelchair is equipped with tilting actuator. It is easy to adjust the seat tilting position 0° to 20°:

1. Press the seat function select button to choose the "Tilt" seating function.
2. The selected seating function will be displayed on the drive/actuator status indicator.
3. Pull the joystick forward or backward from the neutral position.
4. The tilting angle of seat will change accordingly.


6.6 Headrest Adjustment

Comfortable foaming headrest

If your wheelchair is equipped with foaming headrest.

It is easy to adjust the headrest position:

1. Rotate the adjustment screw (1) counterclockwise.
2. Loosen the headrest height adjustment tube.
3. Pull the headrest upward or downward to adjust to the desired height position.
4. Rotate the adjustment screw (1) clockwise to lock the height adjustment tube.
5. Rotate the adjustment screw (2)(3)(4) counterclockwise.
6. Adjust the headrest to the comfortable position you like.
7. Rotate the adjustment screw (2)(3)(4) clockwise to lock.

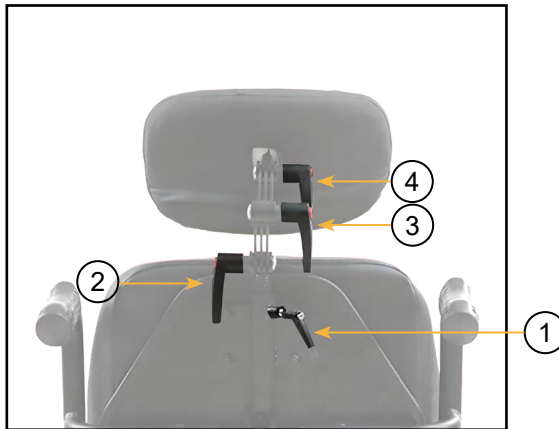7. BASIC TROUBLESHOOTING

WARNING – If the battery symbol light is flashing on the battery gauge then the wheelchair has detected a fault. If this happens, follow the procedure below:

1. Release the joystick, then power the wheelchair off and on. If this does not clear the fault then proceed to step 2 below.
2. Power the wheelchair off and recharge the battery fully. If this does not clear the fault then proceed to step 3 below.
3. Count the number of flashes. The symbol will flash a number of times before a long pause. Count the number of flashes between pauses and report this to your dealer. The flash error codes are listed below for reference.

Flash	Description	Meaning
1	Battery Low	The batteries are running low. • Recharge the batteries.
2	Low Battery Fault	The batteries have run out of charge. Recharge the batteries. Check the battery and associated connections and wiring.
3	High Battery Fault	Battery voltage is too high. This may occur if overcharged &/or travelling down a long slope. If travelling down a slope, reduce your speed to minimize the amount of regenerative charging.

Flash	Description	Meaning
4	Current Limit Time-out or Controller too hot	<p>The motor has been exceeding its maximum current rating for too long.</p> <ul style="list-style-type: none"> • The scooter may have stalled. Turn the controller off, leave for a few minutes and turn back on again. • The motor may be faulty. Check the motor and associated connections and wiring.
5	Park Brake Fault	<p>Either a park brake release switch is active or the park brake is faulty.</p> <ul style="list-style-type: none"> • Check the park brake and associated connections and wiring. • Ensure any associated switches are in their correct positions.
6	Drive Inhibit	<p>Either a Stop function is active or a Charger Inhibit or OONAPU condition has occurred.</p> <ul style="list-style-type: none"> • Disconnect the Battery Charger. • Ensure the joystick is in neutral when turning the controller on. • The joystick may require re-calibration.
7	Speed Pot Fault	<p>The joystick, speed limit pot, SRW or their associated wiring may be faulty.</p> <ul style="list-style-type: none"> • Check the joystick and speed pot and associated connections and wiring.
8	Motor Voltage Fault	<p>The motor or its associated wiring is faulty.</p> <ul style="list-style-type: none"> • Check the motor and associated connections and wiring.
9	Other error	<p>The controller may have an internal fault.</p> <ul style="list-style-type: none"> • Check all connections & wiring.

8. BATTERIES AND CHARGING

Read the battery charging instructions in this manual before charging the batteries. The battery condition meter on the tiller console indicates the approximate strength of your batteries using a color code. From right to left on the meter, green indicates fully charged batteries, yellow a draining charge, and red indicates that an immediate recharge is necessary. To ensure the highest accuracy, the battery condition meter should be checked while operating your Power Chair at full speed on a dry, level surface.


8.1 First charge

Your Power Chair requires two long-lasting, 12-volt, deep-cycle batteries that are sealed and maintenance-free.

They are recharged by the supplied off-board charging system. Charge your Power Chair's batteries for at least 8 to 14 hours prior to using it for the first time. Keep the batteries fully charged to keep your Power Chair running smoothly.

8.2 Charging your Power Chair

The Power Chair can be charged using the charging ports located on the Power Chair or on the battery pack. Plug the charger into the Power Chair or battery pack and then into the electrical outlet. When the batteries are fully charged, remove the charger from the Power Chair.

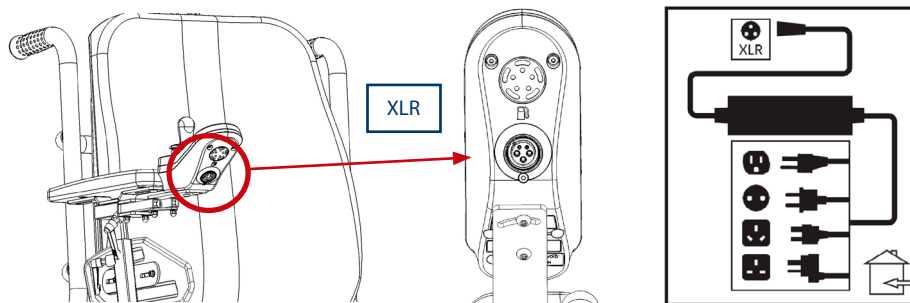8.3 Operating temperatures

Your Power Chair is designed to operate ideally between -10°C ~ 50°C (-14°F to 122°F). In extremely cold conditions, your batteries may freeze depending on the battery charge, usage, and composition of the batteries. In extremely hot conditions, your Power Chair may operate at a slower speed. This is due to the safety mechanism built into the unit to prevent damage to the gearbox and other electronic functions when operating too hot.

8.4 Battery operation and charging

If you use your Power Chair daily, charge its batteries as soon as you finish using it for the day. Your Power Chair will be ready each morning. We recommend that you charge your Power Chair's batteries for at least 8 to 14 hours after daily use. We recommend charging the batteries for an additional 4 hours after the battery charger indicates that charging is complete.

If you use your Power Chair once a week or less, charge its batteries at least once a week for at least 24 hours.

8.5 Charging batteries

 **ATTENTION** – Keep your batteries fully charged and avoid deeply discharging your batteries. Refer to the manual supplied with the battery charger for charging instructions. We recommend charging your batteries for at least 48 continuous hours once per month to improve battery performance and battery life.

The light on your charger indicates different charge status.

Red Light – This means that charging is in operation.

Green Light – This means that bulk charging (95% charge) is complete and the charger is trickle charging the battery.

 **ATTENTION** – There is a charger that inhibits function on your Power Chair. The Power Chair will not run, and the battery condition meter will not operate while the batteries are charging (when the battery pack is being charged on the Power Chair).

 **WARNING** – Removal of grounding prongs can create an electrical hazard. If necessary, properly install an approved 3-pronged adapter to an electrical outlet having 2-pronged plug access.

WARNING – Never use an extension cord to plug in your battery charger. Plug the charger directly into a properly wired standard electrical outlet.

WARNING – Do not allow unsupervised children to play near the Power Chair while the batteries are charging. We recommend that you do not charge the batteries while the Power Chair is occupied.

WARNING – Explosive gases may be generated while charging the batteries. Keep the Power Chair and battery charger away from sources of ignition such as flames or sparks and provide adequate ventilation when charging the batteries.

WARNING – You must recharge your Power Chair's batteries with the supplied off-board charger. Do not use an automotive-type battery charger.

WARNING – Inspect the battery charger, wiring, and connectors for damage before each use. Contact your authorized Provider if the damage is found.

WARNING – Do not attempt to open the battery charger case. If the battery charger does not appear to be working correctly, contact your authorized Provider.

WARNING – Be aware that the battery charger case may become hot during charging. Avoid skin contact and do not place on surfaces that may be affected by heat.

WARNING – If your battery charger has not been tested and approved for outdoor use, then do not expose it to adverse or extreme weather conditions. If the battery charger is exposed to adverse or extreme weather conditions, then it must be allowed to adjust the difference in environmental conditions before use indoors.

9. CARE & MAINTENANCE

Your Power Chair requires a minimal amount of care and maintenance. If you do not feel confident in your ability to perform the maintenance listed below, you may schedule inspection and maintenance at your authorized provider. The following areas require periodic inspection or care and maintenance.

9.1 Tire

Regularly inspect your Power Chair's tires for signs of damage or wear.

9.2 Wheel replacement

Your Power Chair is equipped with PU tires. If you have a damaged or worn tire, the entire wheel must be replaced. Contact your authorized provider for information regarding replacement wheels for your Power Chair.

WARNING – Wheels on your Power Chair should only be serviced/replaced by a qualified technician. Be sure that the key is removed from the key switch and the Power Chair is not in freewheel mode before performing this procedure.

9.3 Exterior surfaces

Bumpers, tires, and trim can benefit from an occasional application of a rubber or vinyl conditioner.

9.4 Cleaning and disinfection

Use a damp cloth and mild, non-abrasive cleanser to clean the plastic and metal parts of your Power Chair. Avoid using products that may scratch the surface of your Power Chair. If necessary, clean your product with an approved disinfectant. Make sure the disinfectant is safe for use on your product before application.

WARNING – Follow all safety instructions for the proper use of the disinfectant or cleaning agent before applying it to your product. Failure to comply may result in skin irritation or premature deterioration of upholstery or Power Chair finishes.

9.5 Battery terminal connections

Make certain that the terminal connections remain tight and are not corroded.

9.6 Wiring harnesses

- Regularly check all wiring connections.
- Regularly check all wiring insulation, including the charger power cord, for wear or damage.
- Have your authorized Provider repair or replace any damaged connector, connection, or insulation that you find before using your Power Chair again.

9.7 Daily checks

With the power turned off, check the throttle. Make sure it is not bent or damaged and that it returns to the neutral position when you release it. Do not try to repair it. See your authorized provider if there is a problem.

Visually inspect the tiller cable. Make sure that it is not frayed, cut, or has any wires exposed. See your authorized provider if there is a problem.

Check for flat spots on solid tires. Flat spots could adversely affect stability.

Inspect the armrests for loose hardware, stress points, or damage. See your authorized provider if there is a problem.

Check the brakes. This test should be carried out on a level surface with at least 3 feet (1 meter) of clearance around your Power Chair.

9.8 Weekly checks

Inspect the controller and charger connectors for corrosion. Contact your authorized provider if necessary.

Check for proper tire inflation if equipped with pneumatic tires. If a tire does not hold air, contact your authorized provider for replacement of the tube.

9.9 Monthly checks

- Check that the anti-tip wheels do not rub the ground when you operate the Power Chair.
- Check for extreme wear on the anti-tip wheels. Replace them as necessary.
- Check for tire wear. See your authorized provider for repair.
- Keep your Power Chair clean and free of foreign material, such as mud, dirt, hair, food, drink, etc.

9.10 Yearly checks

Take your Power Chair to your authorized provider for yearly maintenance, especially if you use your Power Chair on a daily basis. This helps ensure that your Power Chair is functioning properly and helps prevent future complications.

9.11 ABS plastic shrouds

If your Power Chair has a body shroud with a glossy finish, the body shroud has been sprayed with a clear sealant coating. You can apply a light coat of car wax to help it retain its high-gloss appearance. If your Power Chair has a body shroud with a matte finish, use **ONLY** products developed for matte-finish paint. Do not use wax, detail spray, ArmorAll®, or any product made for a glossy paint.

9.12 Axle bearings and the motor/transaxle assembly

These items are all pre-lubricated, sealed, and require no subsequent lubrication.

9.13 Console, charger, and rear electronics

Keep these areas free of moisture. Allow these areas to dry thoroughly if they have been exposed to moisture before operating your Power Chair again.

9.14 Nylon lock nut replacement

Any nylon insert lock nut removed during the periodic maintenance, assembly, or disassembly of the Power Chair must be replaced with a new nut. Nylon insert lock nuts should not be reused as it may cause damage to the nylon insert, resulting in a less secure fit. Replacement nylon insert lock nuts are available at local hardware stores or through your authorized provider.

9.15 Storing Your Power Chair

If you plan on not using your Power Chair for an extended period of time, it is best to:

- Fully charge its batteries prior to storage.
- Remove the battery pack from the Power Chair.
- Store your Power Chair in a warm, dry environment.
- Avoid storing your Power Chair where it will be exposed to temperature extremes.
- Recommended storage temperature: -10°C/-14°F to 50°C/122°F.

WARNING – Operation under high temperature, tires becoming soft.
Operation under very low temperature, user gets freezing.
Operation under very low temperature, tires crack.

- Recommended storage humidity: 30% to 70%.

WARNING – High humidity content may affect the strength of the frame.

Batteries that are regularly and deeply discharged, infrequently charged, stored in extreme temperatures, or stored without a full charge may be permanently damaged, causing unreliable performance and limited service life. It is recommended that you charge the Power Chair batteries periodically throughout periods of prolonged storage to ensure proper performance.

For prolonged storage, you may wish to place several boards under the frame of your Power Chair to raise it off of the ground. This takes the weight off the tires and reduces the possibility of flat spots developing on the areas of the tires contacting the ground.

9.16 Disposal of your Power Chair

Your Power Chair must be disposed of according to applicable local and national statutory regulations. Contact your local waste disposal agency or authorized provider for information on proper disposal of packaging, metal frame components, plastic components, electronics, batteries, neoprene, silicone, and polyurethane materials.

10. SPECIFICATIONS

Model	Felix [®] RT
Type	Power chair, rear wheel drive, Class A
Frame	Rigid blue steel
Max. weight of user	136 kg
Max. speed (driving forward)	10 kmh
Min. braking distance at max. speed	100 cm
Endurance mileage	24V52Ah Lead-acid Battery: 45 km
Maximum height of obstacle	5 cm
Maximum safe slope	10°
Minimum turning radius	90 cm
Minimum width required to reverse	90 cm
L x W x H Dimensions unfolded	108 cm x 58 cm x 121.5 cm
Carton size	102 cm x 72.5 cm x 85 cm
Total mass, battery included	102.4 kg (24V52Ah Lead-acid Battery)
Weight of battery	30 Kg
Effective seat depth	43-49 cm
Effective seat width	43-48 cm
Seat surface height at front edge	45 cm
Height footplate	13-20 cm
Angle seat surface	0-15°
Motor	2 x 340 Watt, electromagnetic brake
Battery	24V52Ah(C20) / TNE12-52
Controller	Dynamic Controller+Lights+Actuator & USB
Protection class	IP X5
Battery charger	Input AC100-240VAC, Output DC24V, 2Amp
Protection class of battery charger	IP X4
Insulation class of battery charger	Class 2
Max. sound level scooter motor	68 dB
EMC compatible according to	ISO 7176-21
Ignitability of upholstery according to	ISO16840-10
Diameter rear wheels (number)	14x3 inch solid
Diameter front wheels (number)	9x3 inch solid
Suspension	Rear suspension
Storage and use temperature	-10 ° C to +50 ° C
Humidity to store and use	30% ~70%

We reserve the right to introduce technical changes. Measurement tolerance $\pm 15 \text{ mm} / 1.5 \text{ kg} / 1.5^\circ$

* The theoretical action radius reduces if the power chair is frequently used on slopes, rough surfaces or to climb kerbs. The maximum driving distance is tested in ideal circumstances according to ISO7176-4.

Speed and range may vary with user weight, type of terrain, battery charge and condition. The information contained herein is correct at the time of publication; Loh Medical® reserves the right to alter specifications.

Disclosure information (ISO)		
	min.	max.
Overall length with legrest	115.5 cm	182 cm
Overall width	59 cm	71 cm
Folded length		85 cm
Folded width		60 cm
Folded height		97 cm
Total mass		102.4 kg
Energy consumption		30 km
Obstacle climbing		4.5 cm
Maximum speed forward		10 kmh
Minimum braking distance from max speed		150 cm
Seat plane angle	-5°	20°
Effective seat depth	43 cm	49 cm
Effective seat width	43 cm	48 cm
Seat surface height at front edge		7.5 cm
Backrest angle	90°	140°
Backrest height		52 cm
Footrest to seat distance	45 cm	57.5 cm
Leg to seat surface angle		110°
Armrest to seat distance	17 cm	29 cm
Minimum turning radius	90 cm	
Maximum occupant mass		136 kg

11. WARRANTY

11.1 Two (2) Years Limited Warranty

Structural frame components, including: seat support tube and welded frames, have two (2) years warranty.

11.2 One (1) Year Limited Warranty

Your Power Chair is fully guaranteed for twelve (12) months from the date of purchase against faults arising due to defects in manufacture or materials. This warranty does not detract from, but is in addition to your legal rights.

All electronic parts and drivetrain, including controllers, battery chargers, differential, motor, wheels and brake, have a one (1) year warranty. Servicing to the controller, battery or battery charger must be carried out by your authorized Loh Medical® Provider. Any attempt to open or dismantle these items renders the guarantee void on that item.

11.3 Not Covered Under Warranty

This guarantee does not extend to those items which may need replacement due to normal wear and tear (tires, belts, bulbs, upholstery, plastic shrouds, motor brushes, fuses, and batteries), or damage to the product caused by misuse or accident for which Loh Medical® or its agent cannot be held responsible. This warranty does not include labor or service calls.

11.4 Batteries

 ATTENTION – Gradual deterioration in performance due to being left in a discharged state, left in cold conditions for long periods of time, or worn out through heavy use is not covered.

Batteries are covered by a six (6) month warranty from the original manufacturer.

11.5 Service Checks and Warranty Service

Warranty service can be performed by an authorized Loh Medical® Provider. Please contact your authorized Loh Medical® Provider for advice on the current cost affecting the service visit.

11.6 Replacement Units

The availability of replacement units and/or spare parts is subject to the discretion of the provider, not the manufacturer. For more information regarding replacement units and/or spare parts, contact your authorized Loh Medical® Provider.

Appendix I :
EMC compliance information 2024-07

Guidance and Manufacturer’s Declaration

Below cables information are provided for EMC reference.

Cable	Max. cable length, Shield/unshielded		Number	Cable classification
AC Power Line	170 cm	Unshielded	1 Set	AC Power
DC Power Line	115 cm	Unshielded	1 Set	DC Power

Important information regarding Electro Magnetic Compatibility (EMC)

This electrical medical equipment needs special precautions regarding EMC and put into service according to the EMC information provided in the user manual; The equipment conforms to this IEC 60601-1-2:2014 standard for both immunity and emissions. Nevertheless, special precautions need to be observed:

The equipment with no ESSENTIAL PERFORMANCE is intended used in Home healthcare environment.

WARNING: Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally”.

The use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

WARNING: Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the model VIRGO series, including cables specified by the manufacturer. Otherwise, degradation of the erformance of this equipment could result.”

WARNING: If the use location is near (e.g. less than 1.5 km from) AM, FM or TV broadcast antennas, before using this equipment, it should be observed to verify that it is operating normally to assure that the equipment remains safe with regard to electromagnetic disturbances throughout the expected service life.

EMI Compliance Table (Table 1)**Table 1 - Emission**

Issue	Compliance	Electromagnetic environment
RF emissions	CISPR 11 Group 1, Class B	Home healthcare environment
Harmonic distortion	IEC 61000-3-2 Class A	Home healthcare environment
Voltage fluctuations and flicker	IEC 61000-3-3 Compliance	Home healthcare environment

EMS Compliance Table (Table 2-5)**Table 2 - Enclosure Port**

Issue	Basic EMC standard	Immunity test levels
		Home healthcare environment
Electrostatic Discharge	IEC 61000-4-2	±8 kV contact ±2kV, ±4kV, ±8kV, ±15kV air
Radiated RF EM field	IEC 61000-4-3	10V/m 80MHz-2.7GHz 80% AM at 1kHz
Radiated RF EM field	IEC 61000-4-3	20V/m 26MHz-2.5GHz 80% AM at 1kHz
Proximity fields from RF wireless communications equipment	IEC 61000-4-3	Refer to table 3
Radiated power frequency magnetic fields	IEC 61000-4-8	30A/m 50Hz or 60Hz

Table 3 –
Proximity fields from RF wireless communications equipment

Test frequency (MHz)	Band (MHz)	Immunity test levels
		Home healthcare environment
385	380-390	Pulse modulation 18Hz, 27V/m
450	430-470	FM, ± 5 kHz deviation, 1kHz sine, 28V/m
710	704-787	Pulse modulation 217Hz, 9V/m
745		
780		
810		
870	800-960	Pulse modulation 18Hz, 28V/m
930		
1720		
1845	1700-1990	Pulse modulation 217Hz, 28V/m
1970		
2450	2400-2570	Pulse modulation 217Hz, 28V/m
5240	5100-5800	Pulse modulation 217Hz, 9V/m
5500		
5785		

Table 4 - Input a.c. power Port

Issue	Basic EMC standard	Immunity test levels
		Home healthcare environment
Electrostatic fast transients/burst	IEC 61000-4-4	± 2 kV 100kHz repetition frequency
Surges Line-to-line	IEC 61000-4-5	± 0.5 kV, ± 1 kV
Conducted disturbances induced by RF fields	IEC 61000-4-6	3V, 0.15MHz-80MHz 6V in ISM bands and amateur radio bands between 0.15MHz and 80MHz 80%AM at 1kHz
Voltage dips	IEC 61000-4-11	0% U_T ; 0.5 cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315°
		0% U_T ; 1 cycle and 70% U_T ; 25/30 cycles Single phase: at 0°
Voltage interruptions	IEC 61000-4-11	0% U_T ; 250/300 cycles

Table 5 - Signal input/output parts Port

Issue	Basic EMC standard	Immunity test levels
		Home healthcare environment
Conducted disturbances induced by RF fields	IEC 61000-4-6	3V, 0.15MHz-80MHz 6V in ISM bands and amateur radio bands between 0.15MHz and 80MHz 80%AM at 1kHz

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