

Shower chair

Shower chair with a height-adjustable aluminum frame and backrest. Made of aluminum tubing and high-quality PVC material. The seat and backrest feature a “U”-shaped opening to facilitate hygiene and provide greater comfort for the user.



CHARACTERISTICS

- Maximum weight capacity: 100 kg.
- Seat width: 19.5 inches (49.5 cm).
- Seat depth: 12 inches (30.4 cm).
- Backrest height: 30 inches (76.2 cm).
- Height from floor to seat: Adjustable from 14 to 17 inches (35.5 to 43.1 cm).
- Main materials: Blow-molded plastic seat and backrest, structural tubes (seat and backrest), triangular knobs, and stainless steel fasteners.
- Safety service life: 2 years from the date of use.



HOW TO USE

1. Before each use, check that all screws and knobs are securely tightened and that the rubber pads on the legs are not worn.
2. If you need to adjust the seat height, make sure the spring-loaded button is fully inserted into the hole until you hear a “click.”
3. Place the chair only on a flat, level surface inside the shower to ensure stability and prevent accidents.
4. Sit down carefully, keeping in mind that the maximum weight capacity is 100 kg.
5. Keep the chair away from extremely high or low temperatures to prevent the plastic and rubber from deteriorating.
6. Replace the non-slip rubber pads on the legs every 6 months, or sooner if you notice they are worn out (simply pull off the old pad and snap the new one into place).

For more information on obtaining this product

Contact your local Loh Medical representative

Find your local representative at www.lohmedical.com

